Asterasys License Activation Instructions

Updated on Jun/2022 – (english version)

In this guide, we will show you how to activate your Asterasys device.

Continue below to get started.


Purchase your license

To activate your device with your license, you will first need to create an account with Medtech Digital. If you already have one, just make sure to use the same login to purchase your new license, so that all your licenses are registered to the same account. Otherwise, create a new one and take note of the email address and password.

1. Go to Medtech Digital [registration page](#).
2. Chose between Login (registered users) or Register (new users).
3. Enter your *username*, a valid e-mail address, and then click **Log in** or **Register**.
4. Check your inbox for Medtech Digital e-mail titled your username to reset your password.
 - If you don't see the e-mail in your inbox, be sure to check your spam/junk folder.

My account

Login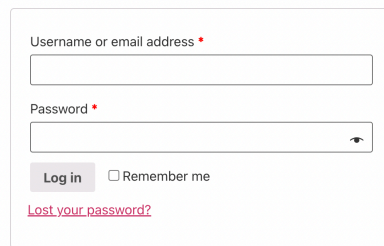


Username or email address *

Password *

Remember me

[Lost your password?](#)

Register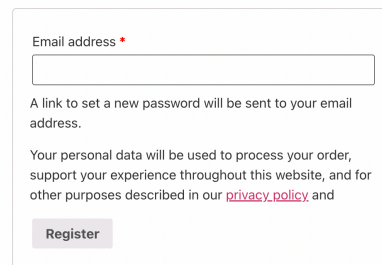

Email address *


A link to set a new password will be sent to your email address.

Your personal data will be used to process your order, support your experience throughout this website, and for other purposes described in our [privacy policy](#) and

Enter your license into your device

Remember: you will need to purchase your device from the manufacturer, local distributor, or re-seller to enter your license. “Device” means a hardware system that runs the operational software and graphical user interface that need to be activated by your license.

To enter your license into the device, make sure to have your device connected to a power source to run it for the first time. Once you run it for the first time you will get one of the following screens, depending on the device you have purchased.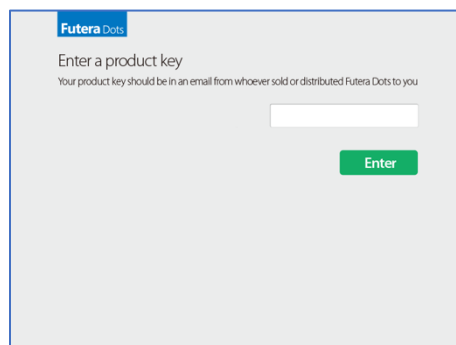


Futera Dots

Enter a product key

Your product key should be in an email from whoever sold or distributed Futera Dots to you

Futera Dots product key screen.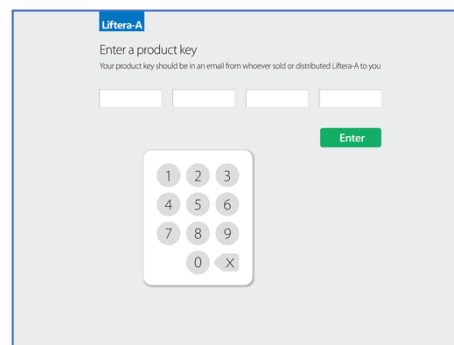

Liftera-A


Enter a product key

Your product key should be in an email from whoever sold or distributed Liftera-A to you


1 2 3
4 5 6
7 8 9
0 X

Liftera-A product key screen.

1. Write down your Product Key sent to you by email. Your product key is in your invoice, and has the following format: XXXX-YYYY-WWWW-ZZZZ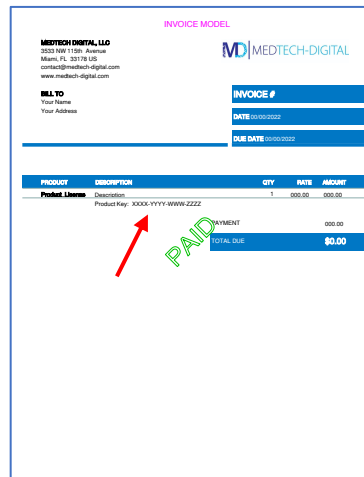

2. Type the Product key on your device booting screen.

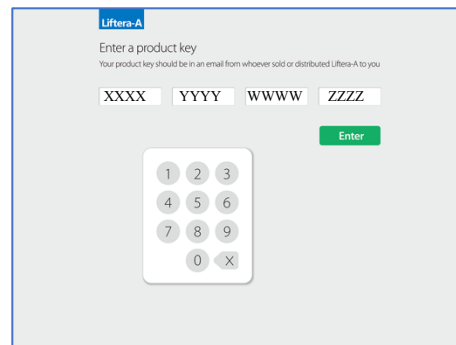3. Wait few seconds until your product is unlocked.
4. Your product is registered and unlocked.

If you have any questions or problems to enter your Product Key, please contact us at contact@medtech-digital.com.

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